medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medi-belgium.be
www.medi-belgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.meditandmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi-hungary.hu
www.medi-hungary.hu

medi Italia S.r.l.
Via Guido Rossa 28
40033 Casalecchio di Reno-
Bologna
Italy
T +39 51 613 24 84
F +39 51 613 29 56
info@medi-italia.it
www.medi-italia.it

medi Medical Support Sdn Bhd
medi representative office Asia
Unit No. B-2-19, Block B, No.2,
Jalan PJU 1A/7A
Oasis Ara Damansara, PJU 1A,
47301 PETALING JAYA
Selangor Darul Ehsan
Malaysia
T: +6 03 7832 3591
F: +6 03 7832 3921
info@medi-asia.com
www.medi-asia.com

medi Middle East
P.O. Box: 109307
Abu Dhabi
United Arab Emirates
T +971 2 6429201
F +971 2 6429070
info@mediuae.ae
www.mediuae.ae

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Polska Sp. z o.o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T +48 32 230 60 21
F +48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

EO06462 / 04.2014

medi

Knöchelbandage

Gebrauchsanweisung. Instructions for use. Mode d'emploi.
Instrucciones de uso. Instruções para aplicação. Istruzioni per
l'uso. Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning.
Návod k použití. Uputa za uporabu. Инструкция по
использованию. Giynime kilavuzu. Instrukcja zakładania.
Οδηγία εφαρμογής. Használati útmutató. Uputstvo za upotrebu.
Інструкція з використання. Navodila za uporabo.

medi. I feel better.

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Important notes

This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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Ankle support

Intended purpose

The product is exclusively to be used for the orthotic fitting of the ankle and only on unbroken skin.

Indications

- Ankle sprains
- Swelling and joint effusion
- Rheumatic and osteoarthritic irritation

Contraindications

Occlusive arterial disease, phlebitis, ulcers, after venous thrombosis; in case of varicosis and leg oedema a compression stocking up to the knee (A – D) should also be worn.

Applying the support

Grasp the upper part of the ankle support (label) and pull it up over the ankle until the heel fits over the integrated recess.

Care instructions

Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Store in a dry place at room temperature. Protect from heat, direct sunlight and moisture.

Material composition

Polyamide, Elastane

Warranty / contractual guarantee

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

Your medi team

wishes you a quick return to full fitness.

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